

# FACE SHIELD SAFETY & REUSE



**TO PERFORM THESE INSTRUCTIONS WEAR A MASK, GLOVES AND WORK IN A SANITISED ENVIRONMENT**

## CLEANING

The following steps are efficient methods of cleaning your shield.



1. Submerge (sink) the Shield parts into the disinfection solution to ensure it reaches 100% of the surface.




2. Use aerosol (spray). Apply the procedure at least twice to reach all of the surface.



3. Remove the remains using a paper towel, avoid using the same towel on multiple shields, avoid cross-contamination. Always wipe in one direction.

## ASSEMBLE & DISASSEMBLE

**WARNING :** ALWAYS DISSASSEMBLE SHEILD BEFORE DISINFECTING

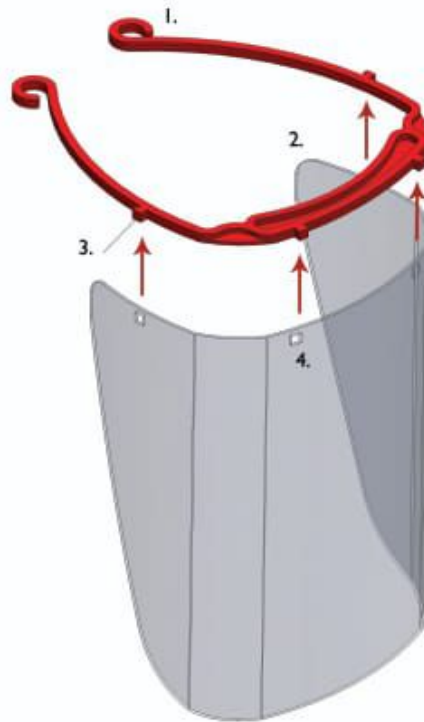

1. Plastic Support
2. OHP Visor
3. Protrusions
4. Punch holes


- a. Place the disinfected OHP visor and plastic support in your disinfected work area, while wearing gloves and a face mask.
- b. Carefully match the holes punched on the OHP visor onto the protrusions on the plastic support from left to right.
- c. Stretch the OHP visor till it fits properly onto the protrusions on the plastic support.
- d. Follow same instructions during disassembly.

## REUSE & DISPOSAL

1. Once assembled make sure your Face Shield fits firmly on your head and does not move.



2. To reuse your Face Shield, remove the OHP Visor from the plastic support, follow the 'Cleaning your Shield' process and replace.

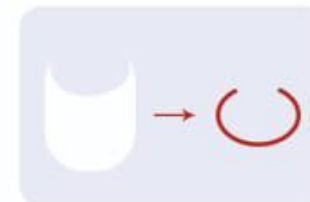

3. In case of irreversible contamination, dismantle the Face Shield and dispose in a closed container designated for hazardous waste



**SAFETY FIRST**



**SCAN QR CODE FOR DETAILED INFORMATION**

